



"All I want for Christmas is my two front teeth."

Donald Yetter Gardner
(Teacher and Song
Writer)

Questions?
Comments?
Concerns?


Contact us !

303 Pontotoc Street
Auburndale, FL 33823

Phone
(863)967-9742
1-866-904-LAVA

Fax
(863)967-2114

E-mail
barbaras@
centraldentalcreations.
com

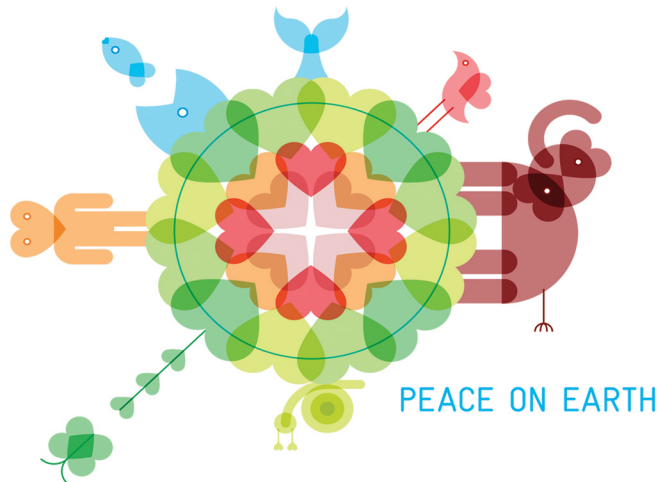

Happy Holidays!

We hope your holiday business is ramping up and we are excited to continue providing you with superior service in 2013 and beyond. We would like to remind you that UPS is not delivering the 24th-25th as well as the 31st-1st. In order to receive a case by December 26th it needs to be in our lab by December 18th. For January 2nd deliver it must be in our lab by December 21st. If you have any specific concerns about a case and shipping times please be sure to call and speak to us directly!


Lava Custom Abutments

Central Dental Creations has been fabricating custom Lava implant abutments for several years now, and have many customers that use them exclusively. If you have not tried them yet, here are some of the advantages to using a Lava Custom Abutment. First, the obvious, it is custom designed, so we can provide the most support and best esthetic restoration possible. It can be shaded the same as all Lava restorations, so you can create a beautiful restoration without worry of metal showing through. Your custom abutment has a metal to metal interface, between the screw and the abutment to allow maximum torque without worry of breakage.

There are a few things we need from you to make the best restoration we can...

- Provide us with the implant type and size – We can do most major brands, call ahead so we can confirm your brand.
- Soft tissue model with removable adjacent teeth. Multiple implants can be left pinned in one section as long as adjacents are removable.
- Any special instruction regarding, design or shading.
- Opposing model and bite.

Now, we can then design and mill a coping, or full contour Lava Plus restoration to complete the case. Or we can send you the abutment for your doctor to seat and take an impression for the final restoration. Custom Abutments cost \$238.00 and take 7 days in lab. Add an additional day for an added restoration. Try them today – You will be glad you did.

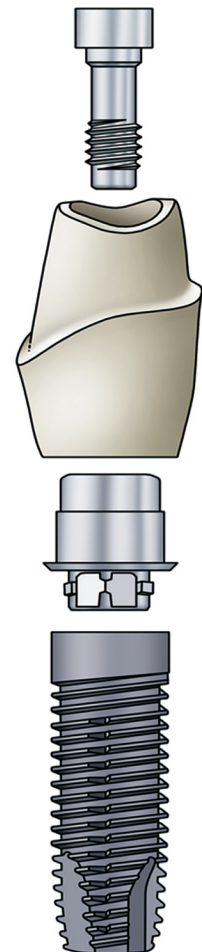



We're on the Web!

See us at:
www.

centraldentalcreations
.com

Like us on Facebook




LAVA PLUS Full
Zirconia

Electronic files \$42
Scan and Design \$62
(single units only)

Email STL Files to:
cdcdesignfiles@
gmail.com

New Lava Products!

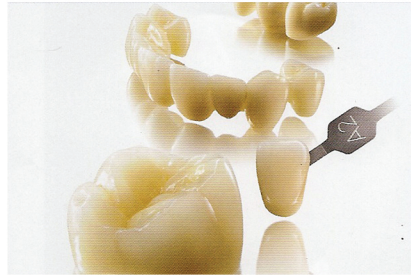

Lava Plus High Translucency Zirconia offers significantly higher translucency without compromising strength.

Excellent match to the VITA Classical Shade Guide and VITA System 3D-Master Shades

Requires 3X less occlusal preparation than lithium disilicate glass ceramic because of its 3X greater strength.



Lava Ultimate Restorative is made with resin nano ceramic technology, resulting in fracture-resistant, natural-looking restorations.

Durability backed by a 10-year warranty
Uniquely adjustable with additive or subtractive techniques

Absorbs chewing forces which reduce stress- especially advantageous for crowns over implants.

Full Contour Zirconia Restoration Finishing / Polish / Glaze

Finish margins, define occlusion and anatomy with high speed handpiece w/ water coolant, using a fine grain diamond bur.

To polish: go over the crown with a silicone white polishing disc to eliminate scratches.

Steam clean before glazing.

To Glaze: Use the glaze from your zirconia porcelain kit OR Emaxx Porcelain Kit

For Porcelain kit:

- Use a thicker mix of glaze, you can use stain in the fissures or white along ridges same as if it was porcelain.
- Increase entry time by a couple of minutes to get zirconia to heat soak.
- Rate of climb 40-45 degrees C per minute.
- Use a longer hold time. 1 minute 20 second to 1 minute 30 seconds. You will have to experiment with your oven and set a cycle.
- Slow cool 2 - 3 minutes
- You can do a second glaze and/or add small amount of correction porcelain on ridges or height of contours to achieve the desired look.

For Emaxx kit:

- Follow manufacturer instructions for applying zirliner, and glaze.

Use DVA Brilliance polishing paste or diamond polishing paste with a bristle brush.