



## Full-Coverage Restorations

### ANTERIOR CROWN PREPARATION

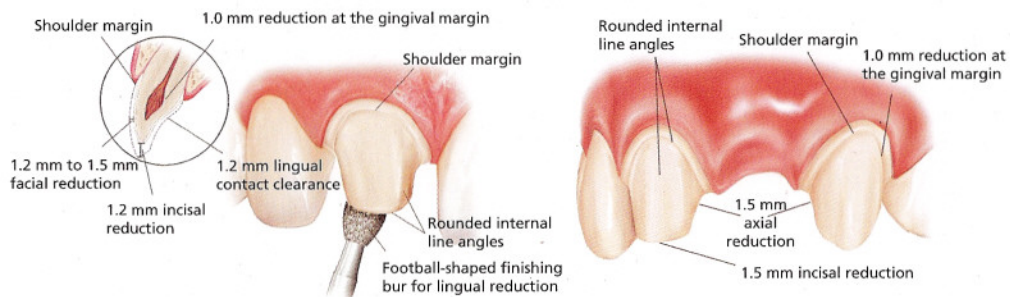


## 3-Unit Bridge Restorations

### 3-UNIT BRIDGE PREPARATION

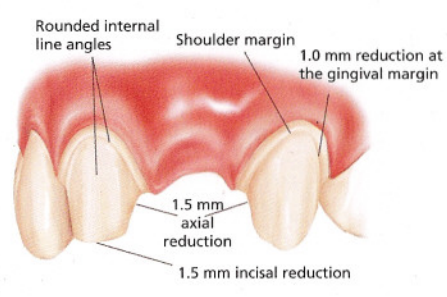


## Inlays/Onlays

### INLAY PREPARATION

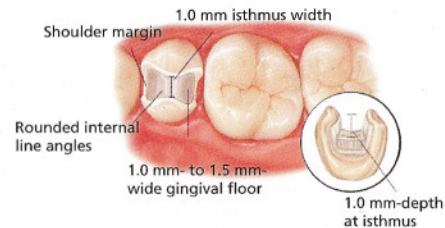


### ONLAY PREPARATION

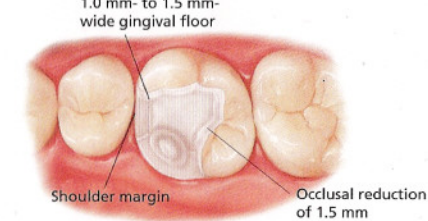

## Veneers

A medium grit, round-ended, diamond bur is used to remove a uniform thickness of facial enamel by joining the depth-cut grooves.

The diamond bur is angled to bevel back the incisal edge.

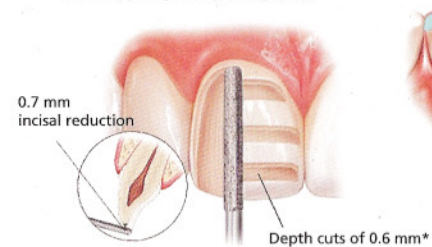



## Thin Veneers

IPS e.max can be pressed to as thin as 0.3 mm for veneers. If sufficient space is present, IPS e.max can be placed over the existing teeth without the removal of any tooth structure. Depending on the case requirements, however, some teeth may need to be prepared to accommodate for the thickness of the ceramic and to ensure for proper contour and emergence profile.

## For IPS e.max® indications


	IPS e.max Ceram/ZirPress	IPS e.max ZirCAD	IPS e.max CAD/Press
<b>Anterior</b>			
<b>Crown</b>		✓	✓
<b>Veneer</b>	✓		✓
<b>Bridge</b>		✓	✓
<b>Posterior</b>			
<b>Crown</b>		✓	✓
<b>Bridge</b>		✓	✓

## 3-Unit Bridge Restorations 3-UNIT BRIDGE PREPARATION

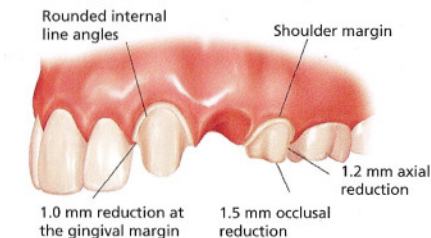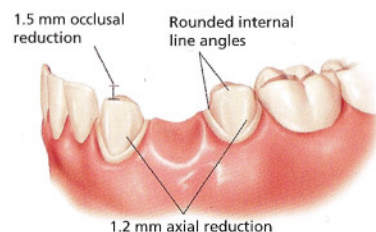


## Full-Coverage Restorations

### POSTERIOR CROWN PREPARATION

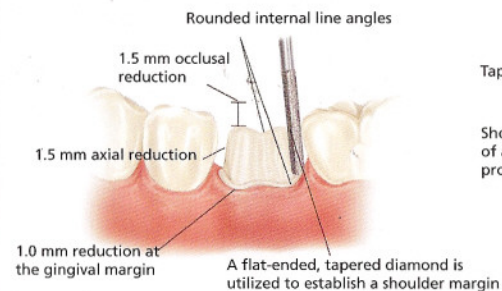

### CONVENTIONAL CEMENTATION PREPARATION

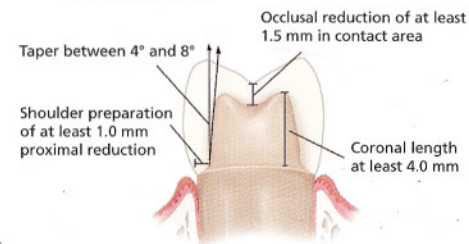

100% CUSTOMER SATISFACTION  
GUARANTEED!

[ivoclarvivadent.com](http://ivoclarvivadent.com)

Call us toll free at 1-800-533-6825 in the U.S., 1-800-263-8182 in Canada.  
598206 Rev. 1 4/10 ©2010 Ivoclar Vivadent, Inc. IPS e.max is a registered trademark of Ivoclar Vivadent.

**ivoclar**  
**vivadent**  
passion vision innovation